

Simple spot check

Valid specimen:



Allow a sufficient quantity of blood to soak through to completely fill the preprinted circle on the filter paper. Fill all required circles with blood. Do not layer successive drops of blood or apply blood more than once in the same collection circle. Avoid touching or smearing spots.

Invalid specimen and possible causes:



1. Specimen quality insufficient for testing.

Possible causes:


- Removing filter paper before blood has completely filled circle or before blood has soaked through to second side.
- Applying blood to filter paper with a capillary tube.
- Allowing filter paper to come into contact with gloved or ungloved hands or substances such as hand lotion or powder, either before or after blood specimen collection.



2. Specimen appears scratched or abraded.

Possible causes:

- Applying blood with a capillary tube or other device.

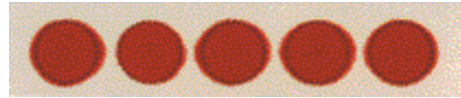

3. Specimen not dry before mailing.

Possible causes:

- Mailing specimen before drying for a minimum of four hours.



4. Specimen appears supersaturated.

Possible causes:

- Applying excess blood to filter paper, usually with a device.
- Applying blood to both sides of filter paper.




Invalid specimen and possible causes:



5. Specimen appears diluted, discolored or contaminated.

Possible causes:


- Squeezing or “milking” of area surrounding the puncture site.
- Allowing filter paper to come into contact with gloved or ungloved hands or substances such as alcohol, formula, antiseptic solutions, water, hand lotion or powder, etc., either before or after blood specimen collection.
- Exposing blood spots to direct heat.

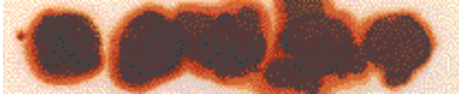

6. Specimen exhibits serum rings.

Possible causes:

- Not wiping alcohol from puncture site before making skin puncture.
- Allowing filter paper to come into contact with alcohol, hand lotion, etc.
- Squeezing area surrounding puncture site excessively.
- Drying specimen improperly.
- Applying blood to filter paper with a capillary tube.

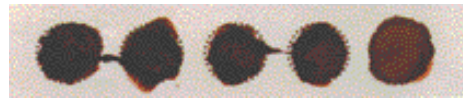

7. Specimen appears clotted or layered.

Possible causes:

- Touching the same circle on filter paper to blood drop several times.
- Filling circle on both sides of filter paper.



8. No blood.

Possible causes:

- Failure to obtain blood specimen.

www.gelifesciences.com/whatman

GE Healthcare UK Limited
Amersham Place
Little Chalfont
Buckinghamshire, HP7 9NA, UK



GE, imagination at work and GE monogram are trademarks of General Electric Company.

Whatman is a trademark of GE Healthcare companies.

Information provided by The New York State Department of Health.

© 2009-2010 General Electric Company – All rights reserved.
Previously published Nov. 2009.

All goods and services are sold subject to the terms and conditions of sale of the company within GE Healthcare which supplies them. A copy of these terms and conditions is available on request. Contact your local GE Healthcare Representative for the most current information.

Björkgatan 30
751 84 Uppsala
Sweden

GE Healthcare Europe, GmbH
Munzinger Strasse 5, D-79111 Freiburg
Germany

GE Healthcare Bio-Sciences Corp.
800 Centennial Avenue
Piscataway, NJ 08855-1327
USA

GE Healthcare Japan Corporation
Sanken Bldg., 3-25-1, Hyakunincho
Shinjuku-ku, Tokyo 169-0073
Japan